

Treatment Menu Quick Setup

1

To power the unit on, pull the green power button out.

Once the system boots to the Cytowave screen, press the enter/select button in the middle of the keypad .



2

MAIN MENU

At the main menu screen, select Treatment.



3

In the applicator select menu, select the appropriate applicator configuration for the treatment cycle.



4


Ensure the correct number of applicators are attached before starting the treatment cycle. Incorrect applicator selection could induce the applicator to exceed normal operational temperatures.



5

Single Foreleg Treatment Selections

- | | |
|---------------------|------------------------------|
| 1. Back: | 4. Hocks: |
| 2. Ligament/Tendon: | 5. Maintenance/Inflammation: |
| 3. Bucked Shins: | 6. Fracture: |

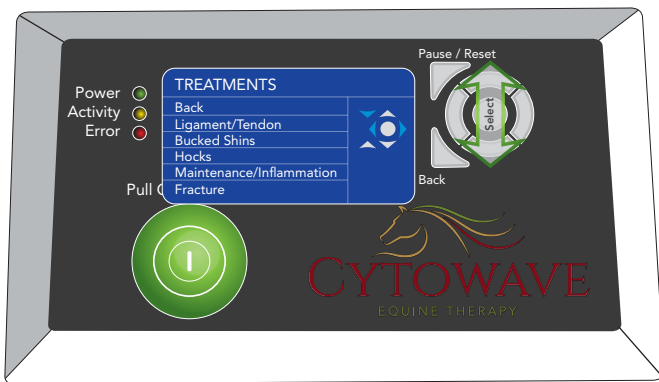

6

Dual Foreleg Treatment Selections


- Ligament/Tendon:
- Bucked Shins:
- Maintenance/Inflammation:



7

Single Hock Treatment Selections

- | | |
|---------------------|------------------------------|
| 1. Back: | 4. Hocks: |
| 2. Ligament/Tendon: | 5. Maintenance/Inflammation: |
| 3. Bucked Shins: | 6. Fracture: |

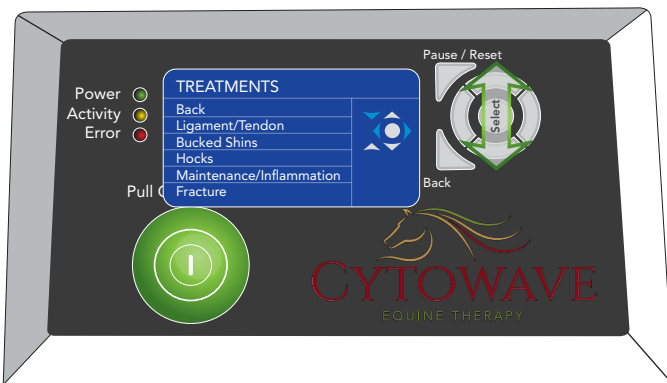

8

Dual Hock Treatment Selections

- | | |
|---------------------|-------------------------------|
| 7. Back: | 10. Hocks: |
| 8. Ligament/Tendon: | 11. Maintenance/Inflammation: |
| 9. Bucked Shins: | |

