

Covetrus[®] ProMAX IV Fluid Warmer



This guide provides a quick reference for operating the Covetrus ProMAX IV Fluid Warmer.

Please refer to the User Manual for more detailed operations and settings.



Pole Clamp and Carabiner

INSTRUCTIONS FOR USE

- If using on a pole – Lower the Pole clamp on the back of the fluid warmer and tighten securely on the pole. See figure A
- If hanging, use the provided carabiner and sling. See figure B

Figure A

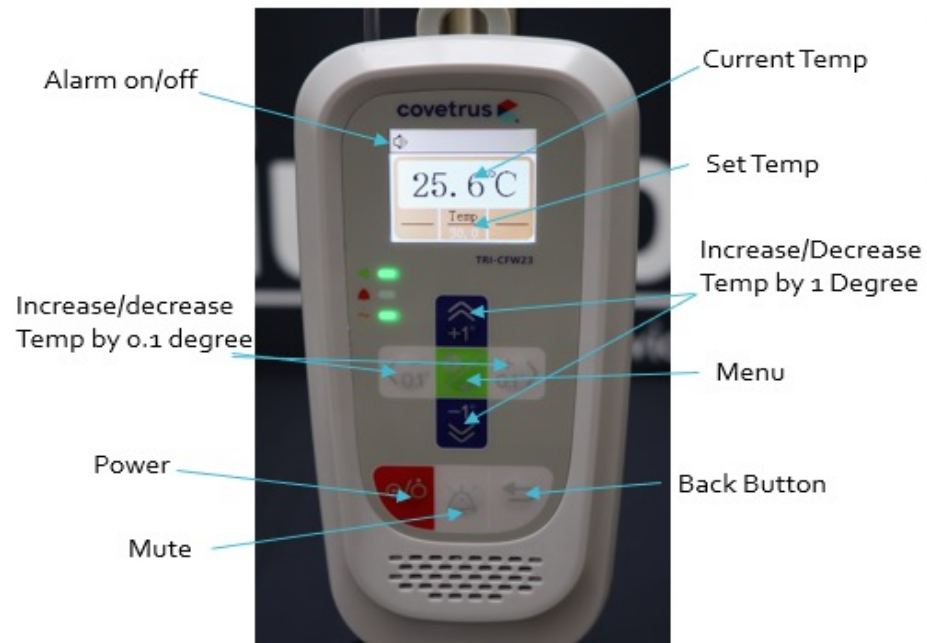


Figure B




Basic Operation

- Fill the IV line with fluid and remove any air in the line. Open the door and install the IV line. The IV line should be installed towards the end of the line, leaving at least 6 inches of IV line from the patient site. Up to 2 IV lines can be installed in the warmer. (IMPORTANT – DO NOT LEAVE IV LINE IN FLUID WARMER WITH NO FLUID FLOWING)
- Press and hold Power Button for 2 Seconds to Power on (IMPORTANT – DO NOT TURN ON WITH NO FLUID FLOWING IN IV LINE)
- Once turned on, the Fluid Warmer will begin working towards the Set Temp. Adjust temperature by increments of 0.1 or 1



Settings

To access the Menu, press the center green Menu/Enter button

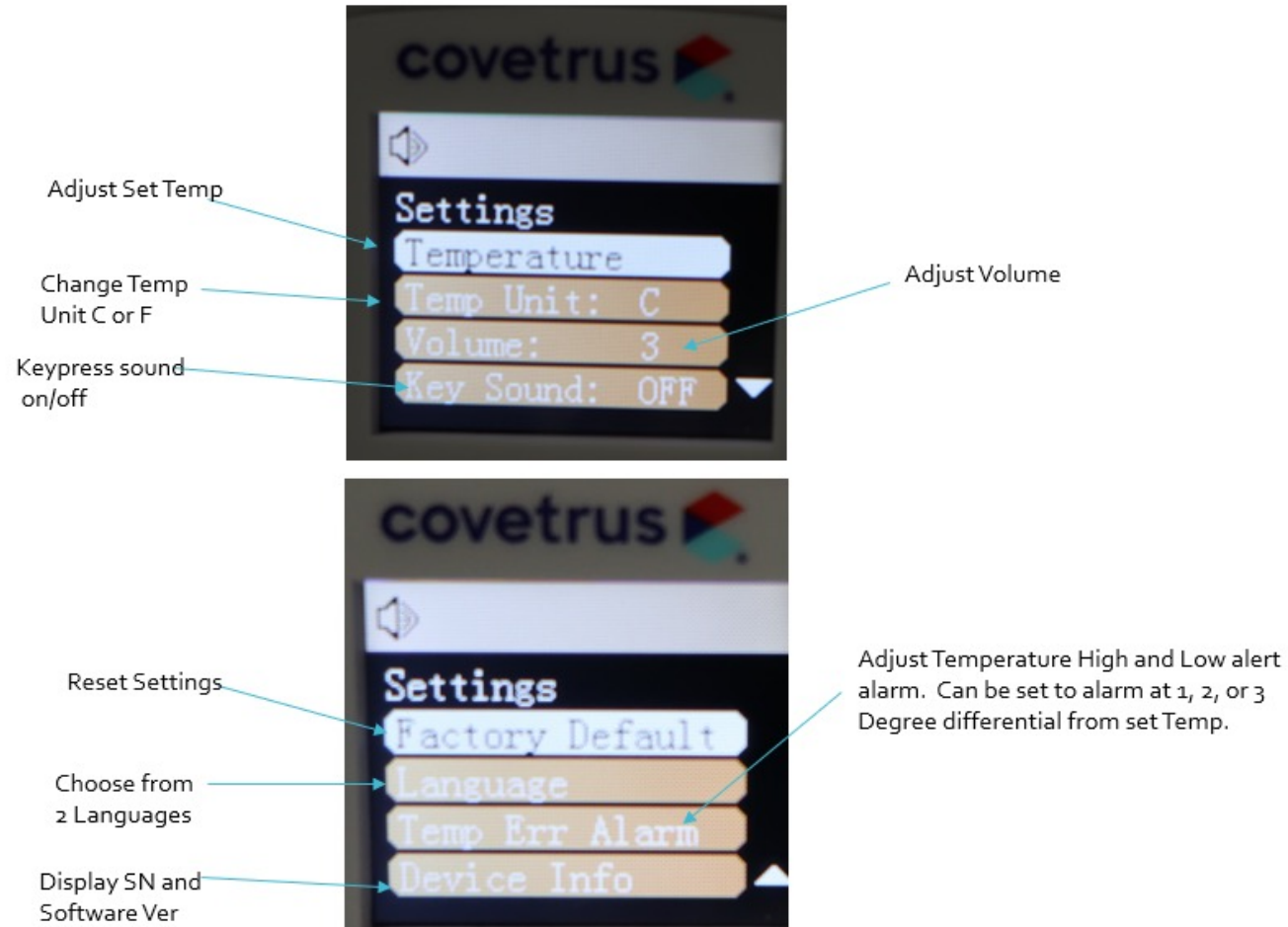

The image shows two screenshots of the device's settings menu. The top screenshot shows the 'Settings' menu with options: 'Temperature', 'Temp Unit: C', 'Volume: 3', and 'Key Sound: OFF'. The bottom screenshot shows the 'Settings' menu with options: 'Factory Default', 'Language', 'Temp Err Alarm', and 'Device Info'. Callouts with arrows point to specific menu items and their functions.

- Adjust Set Temp (points to Temperature)
- Change Temp Unit C or F (points to Temp Unit: C)
- Keypress sound on/off (points to Key Sound: OFF)
- Adjust Volume (points to Volume: 3)
- Reset Settings (points to Factory Default)
- Choose from 2 Languages (points to Language)
- Display SN and Software Ver (points to Device Info)
- Adjust Temperature High and Low alert alarm. Can be set to alarm at 1, 2, or 3 Degree differential from set Temp. (points to Temp Err Alarm)



Service Information

All Warranty and Non-Warranty Repair Services will be performed by Triumph Medical Services. When sending your pump in for repair, please go to the following link and fill out the Veterinary RMA Request form: www.triumphmed.com/vet-rma-request. The shipping address will be provided when your RMA confirmation is emailed to you. For any questions, call 888.962.0662, ext 3





Questions?
(855) 724-3461

Distributed by:
Covetrus® North America
400 Metro Place North
Dublin, OH 43017
[covetrus.com](https://www.covetrus.com)

© 2023 Covetrus, Inc. All rights reserved. No copying without consent.



Reorder #080699

Made in China
AH-ProMAXIVFluidWarmer-01
REV: 0623